







¡Discover in action!

Scan this QR to apply in smartphone or tablets.

#### General Characteristics

HIGH POWER DIODE LASER AND INTENSITY: 1.200W. / 200A.

6 DIODES (GERMAN GOLD BARS 200W).

WORKING TEMPERATURE FROM -5 ° UNTIL -10 °C.

WE DELIVER THE ENERGY BY DEMAND AND WITHOUT CAPACITORS.

DOUBLE POWER SUPPLY.

PULSES AND POWER CAN BE CONFIGURED.

OPTIMUM SPOT FOR MULTIZONE AREAS.

SMART SOFTWARE WITH REMOTE CONTROL.

REMOVABLE HAND PIECE.

HIGHT SPEED FROM 11Hz TO 15Hz.

SWEEP IN SHORT AND LONG PULSE.

#### Diode Laser

DIODE TYPE Semiconductor Class 4

OUTPUT POWER 1.200W. / 200A.

SPECTRUM 808 nm.

SPOT SIZE 10x12mm

PULSE TERM From 15ms a 400ms

SHOOTING SPEED From 1 to 15Hz

FLUENCE up to 40J/cm<sup>2</sup>

WINDOW GLASS Sapphire

#### **Temperature**

TIP COOLING Ethylene Glycol and distilled water

DIODE COOLING Ethylene Glycol and distilled water.

### Communications

SCREEN TYPE A capacitive10"touch Screen LCD.

COMMUNICATIONS Wifi and ethernet\*

\* Optional Characteristic in on request manufacturing lasers

### **Specifications**

INPUT VOLTAGE 220~240V

INPUT FREQUENCY 50/60Hz

CONSUME POWER 1400 VA.

DOWNLOADS PROTECTION Appliance Class 1

WEIGHT 32 Kg.

DIMENSIONS 500 x 540 x 420

### Configurations

SHORT PULSES FROM 15 TO 30 MILLISECONDS
LONGUE PULSES FROM 100 TO 400 MILLISECONDS
SWEEP MODE FOR SHORT AND LONGUE PULSES.
REPETITION PULSE FROM 1 TO 15 HZ

## Visible results from the First Session and on...

Perfect results on customer's Fitzpatrick scale.

Ópphalo LS-1200 works in short pulses of 15 and 30 milliseconds, below the thermal relaxation time of the hair in order to get a thermal damage in the germ cells which are responsible for the hair growth. The efficiency is the same, although we should increase the number of sessions with high melanin density skins (types IV, V and VI in Fitzpatrick scale) in which we will work in long pulses, from 100 to 400 milliseconds.



Type I Very white skin



Type II White skin



Type III Fair Brunette



Type IV



Type V Tanned Brunette Very Dark Skin



Type VI

# High Speed

Works very effectively in blond and thin hair

Ópphalo LS-1200 works from 1HZ (one shot per second) up to 15 HZ (15 shots per second) in short pulse mode (short sweep). For blond and thin hair, this mode has the maximum efficacy and safety.

# Spanish R&D, all in its technology

#### Depilice® patented technology

The special cooling system inside (one and only gas cooling device ever made) lets you to maintain a operating temperature up to -10°C, so the patient can perceive the maximum comfort, protection from a disturbing, uncomfortable feeling of pain.

Ópphalo LS-1200 has no deposits or external must carry around elements, for the comfort of , making it easy to transport or to move around.



## And more:

One and Only Two Seperate Power Supply = Immediate Energy Delivery

Without capacitors, generating its own direct energy.





# High-Capacity, Color Touch Screen

Because nobody knows your customer's skin and hair type better than you, with Ópphalo LS-1200, you set your own shooting parameters, so you get best results and maximum customer satisfaction.

It comes with a 10" LCD Touch Screen, colorful and simple use monitor, so you can preset the treatments and more generic photo types. Remote connection (optional), you can record sessions, statistics and shot settings to control Opphalo LS 1200 any where in the world.

The touch-screen could also be used with surgical-gloves





# THE WARRANTY OF A GREAT COMPANY



+34.900.103.260

www.Opphalo.com

hablemos@opphalo.com

C/ Etileno 2, 47012 Valladolid · SPAIN